

Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) Antigen Detection Kit (Colloidal Gold-Based)

REF	Specifications
C8011CH	1 test/kit
C8010CH	5 tests/kit
C8002CH	20 tests/kit

- A rapid antigen test for the detection of SARS-CoV-2 in nasal swab.
- For self-testing use.
- Please read the instructions carefully before testing.

PREPARATION

- Remove secretions from the surface of the anterior nasal cavity before taking the sample. Wash your hands. Make sure they are dry before testing.
- Read the instructions.
- Check the expiration date printed on the foil pouch of the cassette.
- Open the foil pouch and take the cassette.

TEST PROCEDURE

- Open the package for the cassette, and take out the cassette. Don't touch the tips of the cassette.
- Insert the sample (immersion the sample buffer), make sure the sample is fully immersed in the sample buffer.

INTERPRETATION

If the result is positive, the test line (T) will be visible. If the result is negative, only the control line (C) will be visible.

Biohazard

生物安全袋

FOR SELF-TESTING

Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) Antigen Detection Kit

(Colloidal Gold-Based)

1 Test/Kit

10 MINUTE

FOR SELF-TESTING

Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) Antigen Detection Kit

(Colloidal Gold-Based)

1 Test/Kit

10 MINUTE

Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) Antigen Detection Kit

(Colloidal Gold-Based)

1-Test Cassette

LOT

EC REP

Obelis s.a.
Bd General White 53 1030 Brussels, Belgium
Tel: +32(2)732-59-54 Fax: +32(2)732-60-03
E-mail: mail@obelis.net

SARS-CoV-2 Ag

C T

CE 1434

IVD

Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) Antigen Detection Kit

(Colloidal Gold-Based)

1-Test Cassette

LOT

EC REP

Obelis s.a.
Bd General White 53 1030 Brussels, Belgium
Tel: +32(2)732-59-54 Fax: +32(2)732-60-03
E-mail: mail@obelis.net

SARS-CoV-2 Ag

C T

CE 1434

IVD